

Method of Packing of Sputum Samples for transportation to IRL

Microbiologist- IRL

STDC-Ahmedabad

Gujarat

Materials required

- Diagnostic Sputum transportation BY cold chain in Thermocol box (supplied by IRL)
- Card box container for follow up specimens.
- Diagnosis Sample in Falcon tube/Follow up in CPC Mac Cartney bottle
- Ice gel pack (prefrezed at -20 degree for 48 hours.)
- Annexure I & II Forms
- Polythene bags
- Tissue paper roll as absorbent
- Parafilm tape for Falcon / Mac carteny bottle.
- Brown tape for packaging box.
- Permanent Marker Pen
- Labels
- Bio Hazard logo
- Scissors
- Spirit swab

Filled up Annexure I and II & enter the same Patient's information in Annexure III at DTC

Request for Culture and Drug Sensitivity Testing

ICMR-DCTC will release these reports free of cost to all DTCs. DTCs should send request to DCTC, 2010, New Delhi with necessary documents and samples to the DCTC, 2010, New Delhi. For more information visit www.dctc.gov.in

Name and address of referring health facility: Prakash Hospital, Panchsahi, N.C.E.

ICMR-DCTC address: 2010, New Delhi

Patient Name: Prakash, Prakash Singh Age: 35 Sex: M

Address: Prakash Hospital, Panchsahi, N.C.E.

Diagnosis: Pre-treatment:

Follow-up: (Specify month):

Any other:

Signature of DTC: (Signature)

Street results: Lab Serial No. DMC 4335 (101) Positive (grading)

| Request for Culture - DTC Request for diagnosis and follow-up cultures | | | |
|--|------------------|---------------------------------|-------------------------------------|
| Name of Patient | Reason / History | Name of DTC, DTC and DTC Centre | Specimen No. (Specify Cat. I or II) |
| 101 | Chronic cough | ICMR-DCTC | 101 |
| 102 | Chronic cough | ICMR-DCTC | 102 |
| 103 | Chronic cough | ICMR-DCTC | 103 |
| 104 | Chronic cough | ICMR-DCTC | 104 |
| 105 | Chronic cough | ICMR-DCTC | 105 |
| 106 | Chronic cough | ICMR-DCTC | 106 |
| 107 | Chronic cough | ICMR-DCTC | 107 |
| 108 | Chronic cough | ICMR-DCTC | 108 |
| 109 | Chronic cough | ICMR-DCTC | 109 |
| 110 | Chronic cough | ICMR-DCTC | 110 |


ICMR-DCTC will release these reports free of cost to all DTCs. DTCs should send request to DCTC, 2010, New Delhi with necessary documents and samples to the DCTC, 2010, New Delhi. For more information visit www.dctc.gov.in

PARAFILM TAPE FOR WRAPPING THE FALCON TUBE








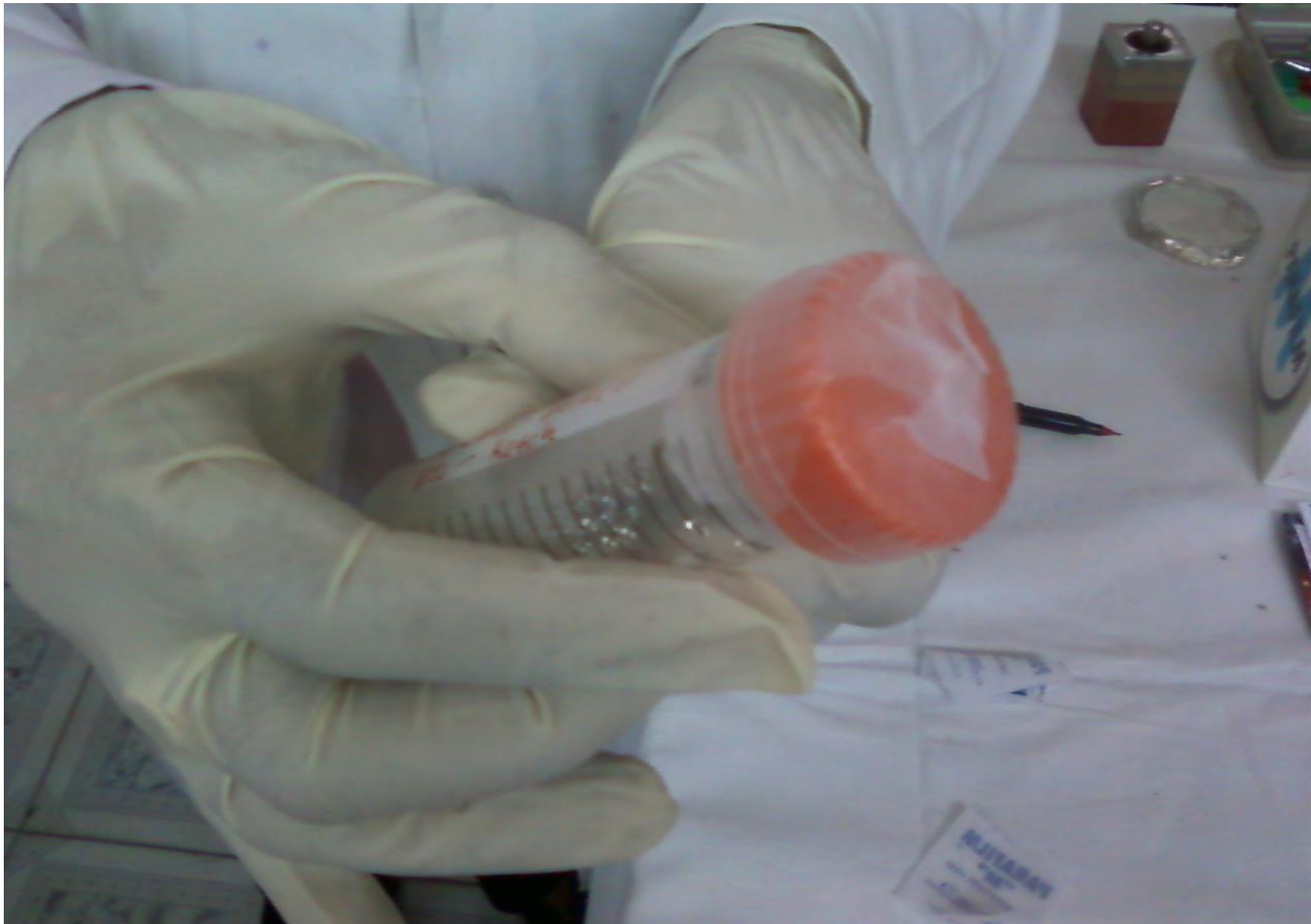

On tube ,label following information
full name of patient date of sample collection
specimen type(A/B) name of DTC/DMC/TU

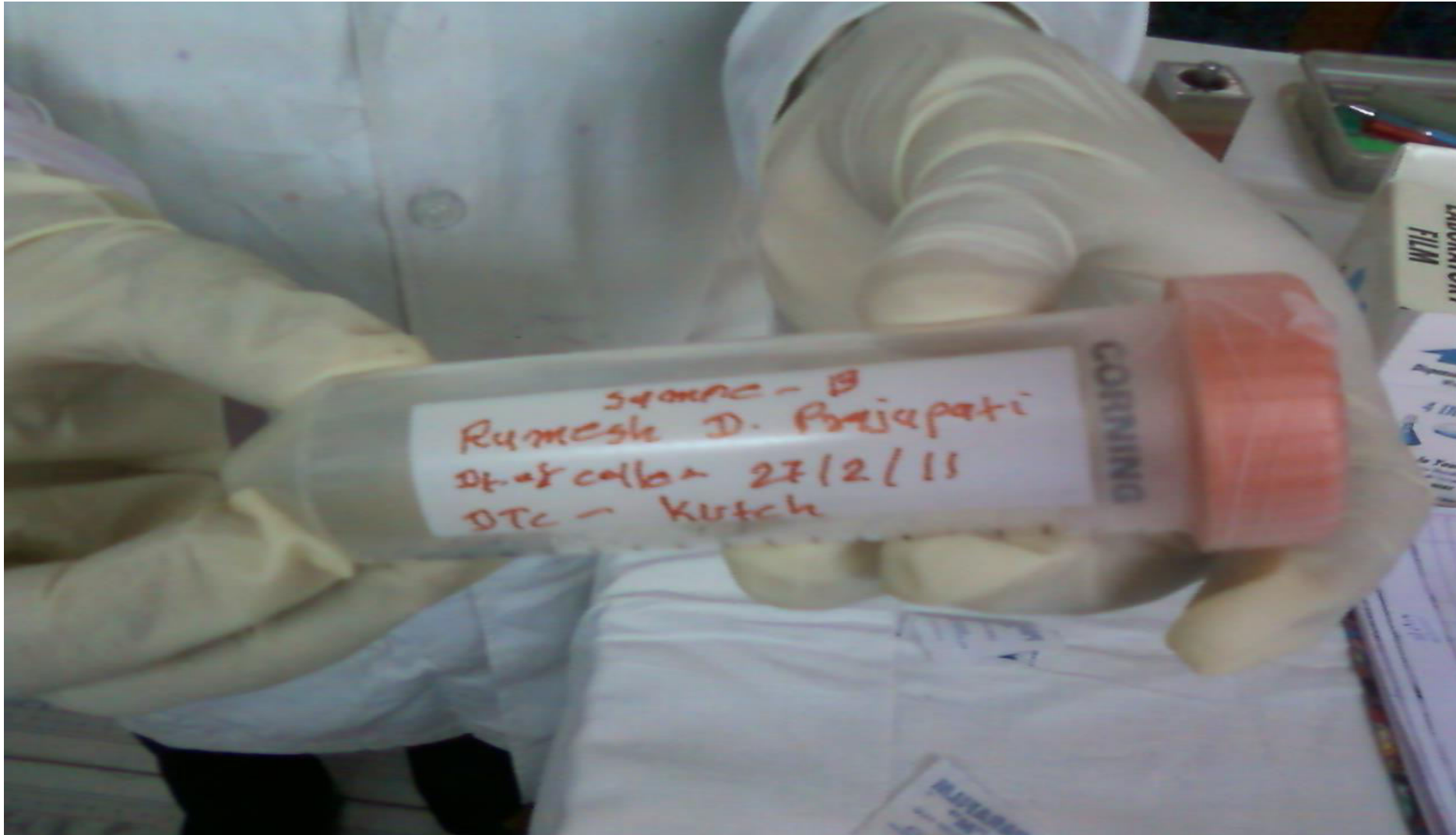

KEEP ONE PREFREEZED (-20 DEGREE FOR 48 HOURS) ICE GEL PACK IN THERMOCOL BOX.



KEEP FALCON TUBES WITHOUT WRAPPING WITH TISSUE PAPER IN BOX.



KEEP 4 FALCON TUBES FOR 2 PATIENTS IN BOX



KEEP ANOTHER PREFREZED ICE GEL PACK ON FALCON TUBES.



KEEP ANNEXURE- I & II IN POLYTHENE BAG AND KEEP IN BOX.



WRAP THE BOX WITH BROWNTAPE/ ADHESIVE TAPE WITH BMW LOGO and ADDRESS .



FOR followup samples pls. send samples in CPC
(supplied by IRL) Mac Macrteny bottles.



Wrap the PARAFILM tape on the cap of bottle / Mac Cartney bottle)



On bottle, label following information

Full Name of the patients

Date of collection of sample

Specimen type (A / B)

Name of DTC



Wrap the entire bottle with toilet tissue paper roll, as absorbent in case of leakage



Put the entire wrapped bottle in a polythene bag and fold the bag



Put Annexure I & II in another polythene bag and fold the bag
Now wrap the form contain bag along with packed bottle



Label the card box container with following information

Full name of patient

Date of collection

Specimen type (A / B)

Name of DTC

BMW logo



Insert the packed bottle along with the forms in Card box container & mark an arrow indicating –This Side Up



Wrap adhesive tape on card box container and keep all cardbox container in another box and send to IRL

